INNOVATIVE TOOLS FOR

*cytogenetics*

*Chromosome  
kit and medium  $\Phi$*

EK AMT-200  
EK AMT-500  
EK AMTB-100  
EK AMTB-500

**INSTRUCTION  
FOR USE**

CE

IVD

**Euro**  **clone**<sup>®</sup>

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## CHROMOSOME KIT P :INSTRUCTIONS FOR USE

1. Aseptically add 0.5 ml of whole blood to the tube; tighten the cap and mix
2. Incubate horizontally at  $37 \pm 2^\circ \text{C}$  for 72 hours (CO<sub>2</sub> not required)
3. Add 1-2 drops of Colcemid and incubate for 4 hours
4. Centrifuge at 2000 rpm for 3-4 minutes
5. Discard supernatant, leaving about 0.5 ml of pellet material
6. Resuspend vigorously (vortexing is recommended)
7. Add 5 ml of hypotonic solution KCl 0.075M\* - Wait 7 minutes at RT (room temperature).
8. Repeat steps 4), 5), 6)
9. Add 5 ml of modified Ibraimov's solution\*^ (92 ml H<sub>2</sub>O+5 ml acetic acid+3 ml methanol)
10. Repeat steps 4), 5), 6)
11. Add 5 ml of pure methanol\*
12. Repeat steps 4), 5)
13. Add 5 ml of Carnoy's fixative solution\* (3:1 methanol-glacial acetic acid)
14. Repeat step 4) and discard supernatant almost completely
15. Resuspend the pellet with a few drops of freshly prepared fixative
16. Drop onto clean, wet slides under climatically controlled conditions°

### *Notes:*

\* Dispensing must be done energetically in order to ensure better uniformity of the cell suspension

^ An homogeneous brownish color and foam formation will indicate that this step has been performed correctly

° The following procedure is strongly suggested to obtain optimal conditions:

a) allow one drop to evaporate and then observe the slide using a phase contrast microscope; if the chromosomes appear well spread, with sharp, clear edges and no cytoplasmic residues, then proceed with the final preparation dispensing three drops on the slide;

b) should you see metaphases with lack of chromosomes ("broken" metaphases) this is due to an excess of humidity: it is advisable to place your slides under a normal lamp turned on, in order to reduce humidity;

c) if, viceversa, the chromosomes are dense and retaining cytoplasm all around, this is because the environment is too dry: you can obviate simply by means of an humidity source, such as a water bath.

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Euroclone S.p.A. - Life Sciences Division